ACCLINO[®] heal Stent

Patienteninformation

Patient information

acandis[®]

ENGINEERING STROKE SOLUTIONS

Acandis GmbH

Theodor-Fahrner-Str. 6

75177 Pforzheim/Germany

www.acandis.com

CE 0297

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1 Product description

The ACCLINO® heal Stent is a self-expanding nitinol stent with a fibrin/heparin coating. The stent has three platinum-iridium radiopaque markers at each end.

2 Indication

The ACCLINO® heal Stent is intended for use with embolisation materials in the treatment of intracranial aneurysms.

3 Information on magnetic resonance imaging (MRI)

Non-clinical tests have shown that the ACCLINO® heal Stent is suitable for MRI. After implantation of an ACCLINO® heal Stent, patients can be safely scanned with a static magnetic field of 3-Tesla. The rise in temperature of the ACCLINO® heal Stent determined in non-clinical tests during MRI imaging for 15 minutes with a 3-Tesla MR system was less than 0.5 °C.

The MRI imaging quality may be affected if the ACCLINO® heal Stent is located in the area of interest. Optimisation of the imaging parameters is recommended.

Please bring this information together with your implant card to all future MRI appointments.

4 Further information

For more information about this implant as well as updates on the information in this leaflet, please visit our website at www.acandis.com/patientimplantinfo.

The ACCLINO® heal Stent remains implanted for life. In case of questions, medical concerns, health symptoms, as well as operating instructions and necessary follow-up examinations, please consult your attending physician or healthcare professional. In case of an emergency, please notify emergency medical services.